Bedrukkingsformaat 170 x 176 mm Printing size 170 x 176 mm Bedruckung Format 170 x 176 mm	Bedrukking full colour Printing full colour Bedruckung Euroskala	Positie bedrukking all-over Printing position all-over Bedruckungsposition Vollflächig
■ Bedrukkings Formaat Printing Surface Bedruckbare Fläche	■ Vouwlijn Folding Line Falzlinie	■ Afloop Bleed Beschnitt
		■ Geen Tekst Plaatsen No Text Here Kein Text Hier
		■ Ingrediënten Ingredients Zutaten

NL

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EN

Pharmaplast s.a.e. Amria, Free zone 23512, Alexandria, Egypt www.pharmaplast-online.com

EC REP E C Rep Ltd, 5 Fitzwilliam Square East, Dublin 2, D02 R744, Ireland Tel: +353 1 2 544 944 Email: info@ecrep.ie

Geocos B.V. Anklaarseweg 15, 7316 MA Apeldoorn, The Netherlands.

REF COLLN4570 LOT15C24 28-7-2027

Instructions for use:

1. Wash the affected area with soap and water and dry thoroughly since oils and moisture reduce adhesion.
2. If the blister has burst apply a mild antiseptic.
3. Peel off the large release liner flap. Apply centrally over the affected area.
4. Peel off the small release liner flap. Smooth over the surrounding skin.
5. Leave the plaster in place until it loosens naturally. Repeat until healing process is complete.

Warnings:

1. Do not use on infected wounds.
2. If you have diabetes, or problems with circulation, ask your doctor or chiropract before using this product.
3. If the blister fails to heal or the surrounding skin becomes red and swollen, consult your doctor or pharmacist.
4. Use only once. Re-use could result in risk of cross-infection and reduced product performance.
5. Do not use the product in case the pouch is open/ damaged.
6. Do not use for bleeding wounds/ third degree burns.
7. Don't use on patient with known sensitivity to the product or any of its components specially Polyisobutenes and Carboxymethylcellulose.

DE

Pharmaplast s.a.e. Amria, Free zone 23512, Alexandria, Egypt www.pharmaplast-online.com

EC REP E C Rep Ltd, 5 Fitzwilliam Square East, Dublin 2, D02 R744, Ireland Tel: +353 1 2 544 944 Email: info@ecrep.ie

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